

# STAY DRY\*

KIMBERLY-CLARK\* STAY-DRY\* Ice Packs

## Directions for Filling



**1** Open clip by pressing locking tab in and pulling up on top bar.

**2** Grasp neck of ice pack with both hands and twist in opposite directions to open.



**IMPORTANT**  
Do Not Overfill



**3** Fill ice pack to red fill line on label (or 2/3 full) and press to remove excess air.

**4** Fold neck of ice pack over bottom blade of clip, align and smooth out wrinkles.



**5** Carefully close top bar of clip and make sure locking tab snaps into locked position to ensure a leak-proof seal.



For more information about our full line of Cold Therapy products, contact your Kimberly-Clark Clinical Specialist.

### General Use Ice-Filled Packs



### Specialty Use Ice-Filled Packs



### COLD RUSH\* Instant Cold Packs



### FLEXI-TEMP\* Gel Pack



\* Registered Trademark or Trademark of Kimberly-Clark Corporation, Roswell, GA 30076.

\*\* #1 selling ice-filled ice pack based on IMS Data for disposable ice-filled ice packs from hospitals in the United States.

© 2002 Kimberly-Clark Corp.

Protection. For life.



**Kimberly-Clark**